



Bedienungsanleitung



Thera-Band Hand Xtrainer, stark, blau


Art.-Nr. 20839

▶▶▶ [zum Produkt...](#)

Hand-und Fingertrainer

▶▶▶ [zur Kategorie...](#)

Hand Xtrainer

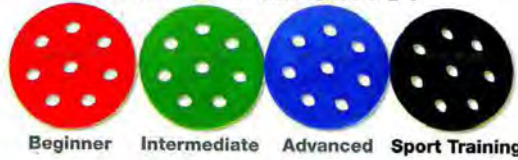

Thera-Band® Hand Xtrainer-Black



Made in USA

The Hygenic Corporation
1245 Home Avenue
Akron, OH 44310 U.S.A
www.thera-band.com

CEpartner4U
Esdorplan 13
3951 DB Maarn
The Netherlands



Hand Xtrainer
THERAPIST RECOMMENDED



CAUTION: Before use consult your healthcare provider. Excessive use may cause hand or forearm fatigue. Consult your physician if soreness or inflammation occurs.

CARE AND CLEANING: Clean with warm soapy water and pat dry with a cloth towel. DO NOT USE any CHEMICAL CLEANERS. Excess tackiness can be eliminated with a light dusting of cornstarch or baby powder.

HOT OR COLD THERAPY: To heat, place in the microwave for no more than 10 seconds. To chill, place in the refrigerator for 1.5-2 hours.

WARNING: Thera-Band Hand Xtrainer is not a toy. Adult supervision is required.

For additional information go to: info.thera.bandacademy.com/xtrainer

Thera-Band®, the Color Pyramid Design™, and the Associated Colors™ are trademarks owned by The Hygenic Corporation or its affiliates and may be registered in the United States and other countries. All rights reserved. Made in USA. U.S. Patent #6,084,027 and 5,798,411. European Patent Convention No.0903377, NO Patent No. 311766, CA Patent No. 2247750, JP Patent No. 3470874, SE Patent No. 9803127-1, AT Patent No. F249499, DE Patent No. P69817957.9, NZ Patent No. 331139 © 2012. P05392 REV 1/12

Hand Crosstrainer
VON THERAPEUTEN EMPFOHLEN



VORSICHT: Vor dem Einsatz fragen Sie Ihren Arzt oder Therapeuten. Übermäßiger Gebrauch kann zu Hand- oder Unterarmmüdigkeit führen. Fragen Sie einen Arzt, wenn Schmerzen oder Entzündungen auftreten.

REINIGUNG UND PFLEGE: Reinigen Sie den Thera-Band Hand Xtrainer mit warmem Seifenwasser und trocknen Sie ihn mit einem Stofftuch ab. VERWENDEN SIE KEINE CHEMISCHEN REINIGER. Wenn die Oberfläche zu klebrig wird, kann sie leicht mit Speisestärke oder Babypuder bestäubt werden.

WÄRME- UND KÄLTETHERAPIE: Wärme: Legen Sie den Xtrainer für max. 10 Sekunden in die Mikrowelle. Kälte: Legen Sie den Xtrainer für 1,5-2 Stunden in den Kühlschrank.

VORSICHT: Der Thera-Band Hand Xtrainer ist kein Spielzeug. Beaufsichtigung durch Erwachsene ist erforderlich.

Zum info.thera.bandacademy.com / XTrainer für weitere Informationen





Sport-Tec GmbH
Physio & Fitness

Lemberger Straße 255
D-66955 Pirmasens

Tel.: +49 (0) 6331 1480-0
Fax: +49 (0) 63 31 14 80-220
E-Mail: info@sport-tec.de
Web: www.sport-tec.de