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## CLASSIFICATION AND USE

### CEILING GRID 250162

Read the instructions and use the product as directed according to its function.

Do not use the product in a way other than its intended purpose.

#### INTENDED USE

Ceiling grid for simple traction systems. Assist with the recovery of arm and leg function in patients who have suffered minor trauma.

#### TYPE OF PATIENT

Patient with non-significant traumatic pathologies to the arms and legs, expected to recover quickly.

#### PLACE OF USE

Hospitals and clinics where space is limited.

#### OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals, physiotherapists and similar All professional users.

The instructions given to the patient by the reference healthcare professional suffice.

#### CONTRAINDICATIONS

Not suitable for use in a way other than the intended purpose.

#### OTHER INDICATIONS

- It must be fastened to the wall by a professional.
- Do not exceed the workload indicated
- Check accessories (ropes and braces) for wear

**Report all serious incidents that occur due to the device. Report the incident to the manufacturer and the competent authority**

**At the end of its life, the product must be disposed of in accordance with local regulations and taken to specific waste collection facilities.**

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**Weight:** 43 kg

**Storage:** dry, dark place, -20°+60°

**Cleaning:** it is generally recommended to clean the device after each use and to carry out the disinfection cycle periodically. To clean the device, wipe its parts with a cloth moistened with warm water. Do not use any agents that could damage the surface of the device through abrasion or overly aggressive treatment.

**Operating temperature:** 0° / +40°

**Expected lifetime:** it is estimated that the device can be used for 10 years.

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✉ info@ferrox.it

SRN: IT-MF-000011243

Under its sole responsibility state that the products:

| Name         | Code   | UDI-DI Basic:      |
|--------------|--------|--------------------|
| CEILING GRID | 250162 | 805347043GRIGLIAJY |

meet all the applicable essential requirements, of Annex I, implemented by the  
(UE) REGULATION 2017/745 OF THE EUROPEAN PARLAMENT AND COUNCIL on Apr 5 th 2017.

Classified (Annex VIII, MDR 2017/745): **Class I (Rule 1)**

Procedure for the assessment of conformity:

**Technical documentation as per annex II and III.**

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The logo for FERROX® features a stylized black silhouette of a person in a dynamic, athletic pose, possibly a runner or a worker, positioned to the left of the word "FERROX" in a bold, black, sans-serif font. A registered trademark symbol (®) is located at the top right of the word.

Stabilimento e sede legale:  
via del Lavoro Z.I.  
31013 Codogné (TV) Italia  
Tel. +39 0438 777099 - 778370  
Fax +39 0438 777091  
P.IVA: IT 03613220262

e-mail: [info@ferrox.it](mailto:info@ferrox.it)  
[www.ferrox.it](http://www.ferrox.it)

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==== [www.sport-tec.de](http://www.sport-tec.de) ====

**Sport-Tec GmbH**  
**Physio & Fitness**  
Lemberger Str. 255  
D-66955 Pirmasens

**Tel.:** +49 (0) 6331 1480-0  
**Fax:** +49 (0) 6331 1480-220  
**E-Mail:** [info@sport-tec.de](mailto:info@sport-tec.de)  
**Web:** [www.sport-tec.de](http://www.sport-tec.de)