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Art.-Nr. ☐ 25384 · Kategorie: ☐ Schlingentisch Zubehör

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Read these instructions carefully before use. Follow the instructions and keep the instructions for use in a safe place.

## INTENDED USE

The Mobilization Wedge is a device intended to provide pressure to the acupressure points on the chest or wrist, adjust the loosening of the various segments of the lumbar and thoracic vertebrae, reducing pain and cervical dysfunction in clients with neck pain.

## TERMS OF DELIVERY

No.	Item	Quantity
1	Mobilization Wedge	1x

## SAFETY INFORMATION

Please note that improper use can lead to injuries. Use the product only for its intended purpose.

The mobilization wedge may only be used by specialist staff.

Only use on a safe and stable counter top.

Contraindication: People allergic to Polyurethane are not allowed to use the product.

## PROPER USE

1. The client lies on a safe and stable surface.
2. With the assistance of healthcare staff the client slowly lays down.
3. The healthcare staff places the device in the treatment position.
4. After treatment the client slowly gets up with the assistance of healthcare staff.

## CARE AND CLEANING

Use a disinfectant suitable for polyurethane for cleaning.

Store under cool, clean, and dry conditions. Avoid excessive heat (over 45°C or 113°F).

## REPORTING OBLIGATIONS

Caution: All serious incidents related to the Mobilization Wedge must be reported to the manufacturer and to the competent authority of the Member State where the user and/or client is established.

## PRODUCT PROPERTIES

	<b>Mobilization Wedge small</b>	<b>Mobilization Wedge large</b>
Item No.	25380	25382
Colour	01-blue, 05-black	01-blue, 05-black
Material	polyurethane	polyurethane
Weight	ca. 140 g	ca. 430 g
Measures (LxWxH)	155x99x50 mm	230x110x88 mm
Measures recess (WxT)	29x30 mm	22x40 mm
Country of origin	Made in Taiwan	Made in Taiwan

**Sport-Tec GmbH**  
**Physio & Fitness**  
Lemberger Str. 255  
D-66955 Pirmasens

**Tel.:** +49 (0) 6331 1480-0  
**Fax:** +49 (0) 6331 1480-220  
**E-Mail:** info@sport-tec.de  
**Web:** www.sport-tec.de