==== [www.sport-tec.de](http://www.sport-tec.de) ====

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**Schröpfgläser ø 2,5-6,5 cm, 5er Set**

**Art.Nr.: 27215**

**▶▶▶ [zum Produkt ...](#)**

**Schröpfglas**

**▶▶▶ [zur Kategorie...](#)**

<b>de</b>	Gebrauchsanweisung Schröpfgläser .....	4
<b>en</b>	User's Manual Glass Suction Cups .....	8

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## User's Manual Glass Suction Cups

Dear Customer, thank you for choosing a KaWe product. Our products are known for their high quality and longevity.

**Please read these instructions thoroughly and carefully before attempting to use this product and heed the care instructions.**

Familiarise yourself fully with this product before using it.

**Application:** The suction cups are only to be used by trained personnel.

**Purpose:** Suction cups are used to massage the skin in certain areas in order to increase well-being.

**Unintended Use / Contraindication of the Products:** Any use other than that described here is not in accordance with the intended use of the instrument. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

**Further Information, Maintenance, and Storage:** When used and stored properly, these products will serve you many years. Always ensure that the glass suction cups are protected against impact: they are breakable!!

**Guarantee:** When used under normal circumstances and with attention to these instructions (User's Manual) we guarantee this product for two years after the date of purchase (except glass breakage). Should you need further information, please contact your dealer!

**Limitation of reuse preparation:** Frequent reuse only affects the instruments in a minor way. The working life of the instrument usually depends on the wear and tear and damage due to use.

**General information:** Please heed the instructions given by the manufacturer of the cleaning/sterilisation device (RDG) as well as those of the manufacturer of the cleaning/disinfecting agents.

### **Instructions:**

**Storage and Transport:** The instruments are to be placed in a tray filled with a suitable cleaning and or disinfecting agent immediately after being used on a patient. Immersing the instruments in a solution keeps residue from drying onto them (protein fixation). It is recommended that the instruments be reprocessed no later than one hour after their use. The instruments are to be transported to the place at which they are reprocessed in a closed instrument tray.

**Cleaning preparation:** No special requirements. Remove the rubber ball from the glass suction cup.

**Manual cleaning:** According to the recommendations made by the Robert-Koch-Institute (RKI), the preferred method for the further reprocessing of the instruments is mechanical sterilisation. Manual cleaning is not recommended. After dry cupping treatment, the suction cups can be disinfected with a surface disinfecting agent.

**Automatic cleaning:** Equipment: CM 310 (Maquet),  
Cleaning agent (neodisher®FA forte 0.4% / neodisher®Z 0.2%) G7828 (Miele)  
Cleaning agent (Mucapur®XL 0.4 %/ Mucapur®Z 0.15 %) WD 390 (Belimed),  
Cleaning agent (Mucapur®AF 0.5 %/ Mucapur®Z 0.1 %)

- 1.** Rinse the instruments with running water immediately before they are mechanically cleaned to insure that no residue from cleaning or disinfecting agents is left on the instruments that could enter the machine.
- 2.** Place the instruments in a suitable instrument stand.
- 3.** Place the instrument stand in the machine such that the instruments are directly hit by the spray jet.
- 4.** Put the cleaning agent into the machine as instructed.
- 5.** Start the Vario TD programme with thermal disinfection. The thermal disinfection programme takes into account both the A0 value and the applicable national regulations (EN/ISO 15883).
- 6.** After the programme cycle has been completed, remove the instruments from the machine and dry them (as recommended

by the RKI, preferably with pressurised air). When using instrument stands, take special care to dry the areas that are hard to reach. **7.** Inspect the instruments to ensure that they are clean and in sound condition with a suitable magnifying device. (Experience has shown that 8-fold magnification allows for optimal visual inspection.) Should there still be leftover residue on the instruments after they have been mechanically treated, repeat the cleaning and disinfection procedures until no more contaminants can be found. **8.** Caution! If the instruments are only cleaned automatically (without verifiable disinfection), a final thermal disinfection of instruments (packed in suitable stands or mesh trays) is necessary.

**Check and Performance test:** Inspect all instruments for damage and wear (broken glass, for example) before using them.

**Packaging:** Individually: One standard polyethylen/tyvek bag can be used. The bag must be large enough for the instrument to fit inside without straining the bag's closure.

**Sterilisation:** The suction cups can be effectively sterilised using the following sterilisation parameters:

- Fractionated pre-vacuum (3-fold)
- Sterilisation temperature: min. 132 °C, max. 137 °C
- Exposure time: minimum of 3 min (full cycle)
- Drying time: 10 min

The ball can only be sterilised by boiling it in water. (Immerse in boiling water for 10 min.)

**Transport and Storage:** The instruments are to be stored and transported such that they are protected from exposure to dust, moisture and contaminants.

**Final recommendations:** The manufacturer ensures that the above-mentioned preparation methods are suitable for the preparation of said groups of instruments for their reuse. The preparer is responsible for ensuring that the reprocessing procedures that are actually performed are done so with the appropriate equipment, materials and personnel in the reprocessing facility and that the desired results are achieved. This will normally require routine inspections of the

validated automated or standardised manual preparation methods. Any deviations from the procedures listed here are to be carefully evaluated by the operator in order to guarantee their effectiveness and guard against possible undesirable outcomes.

**Contact:** Address or telephone number of your dealer or dial +49-7141-68188-0.

**Symbol key:**

: Heed the User's Manual

**Sport-Tec**  
Physio & Fitness

Lemberger Straße 255  
D-66955 Pirmasens

Tel.: 06331/1480-0  
Fax: 06331/1480-220

info@sport-tec.de  
www.sport-tec.de